Introducing the Umbifunnel

An umbilical cord blood collection device -
versatile, safe, efficient and user-friendly.




Instructions for Use - Udhëzime për Përdorim - Инструкции за употреба - 使用说明 - Upute za korištenje -
Návod k použití - Instruktioner for brug - Gebruiksaanwijzing - Kasutusjuhised - Käyttöohjeet -
Guide d'utilisation - Gebrauchsanweisung - Οδηγίες χρήσης - Használati útmutató - Istruzioni per l'uso -
使用方法 - Lietošanas instrukcija - Naudojimo instrukcijos - Упатства за употреба - Struzzjonijiet għall-Użu -
Veiledning for bruk - Instrukcja obsługi - Instruções de Uso - Instrucțiuni de utilizare - Упутство за употребу -
Návod na použitie - Navodila za uporabo - Instrucciones de uso - Instruktioner för användning

Instructions for Use:	2
Udhëzime për Përdorim:	3
Инструкции за употреба:	4
使用说明:	5
Upute za korištenje:.....	6
Návod k použití:	7
Instruktioner for brug:	8
Gebruiksaanwijzing:.....	9
Kasutusjuhised:.....	10
Käyttöohjeet:	11
Guide d'utilisation :	12
Gebrauchsanweisung:	13
Οδηγίες χρήσης:	14
Használati útmutató:	15
Istruzioni per l'uso:	16
使用方法	17
Lietošanas instrukcija:	18
Naudojimo instrukcijos:.....	19
Упатства за употреба :	20
Struzzjonijiet għall-Użu:	21
Veiledning for bruk:	22
Instrukcja obsługi:.....	23
Instruções de Uso:	24
Instrucțiuni de utilizare:.....	25
Упутство за употребу:	26
Návod na použitie:.....	27
Navodila za uporabo:.....	28
Instrucciones de uso:.....	29
Instruktioner för användning:.....	30

Instructions for Use:

- 1) Open the Rotunda Umbifunnel pack.
- 2) Place the Umbifunnel and Umbistand on your workspace.
- 3) Open your blood tube and place the lid on the base of the Umbistand.
- 4) Place the blood tube into the Umbistand.
- 5) Place the Umbifunnel into the blood tube and *gently* push down to secure.
- 6) Place the cut end of the umbilical cord into the Umbifunnel.
- 7) Milk cord until the sample is obtained – Do not fill above the indicator line on the blood tube.
- 8) Close and then invert the tube slowly, turning two (2) to four (4) times to ensure mixing of sample and anti-coagulant.
- 9) Remove gloves and replace with new gloves.
- 10) Tie-off bag containing placenta and cord.
- 11) Dispose of the placenta and cord in the Special Clinical Waste bin .
- 12) Dispose of the Umbifunnel in the Clinical Waste bin.
- 13) Remove blood tube and screw on lid.
- 14) Remove Umbistand and packaging in the General Clinical Waste bin.
- 15) Dispose of gloves.
- 16) Label blood tube.
- 17) Place blood tube in the sample bag.

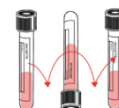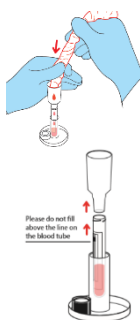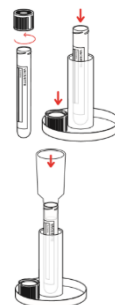Note: It may not be necessary to milk the umbilical cord into the Umbifunnel, natural flow may be sufficient.

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